



**WARNING**


This Quick Reference does not replace the Owner's Guide for your Trak® Male Fertility Testing System. The Owner's Guide contains important handling instructions and additional information.

[trakfertility.com](http://trakfertility.com)

**01**

**Collect a Sample**

Collect sample in provided cup. Place lid on cup and swirl for **at least 15 seconds**.

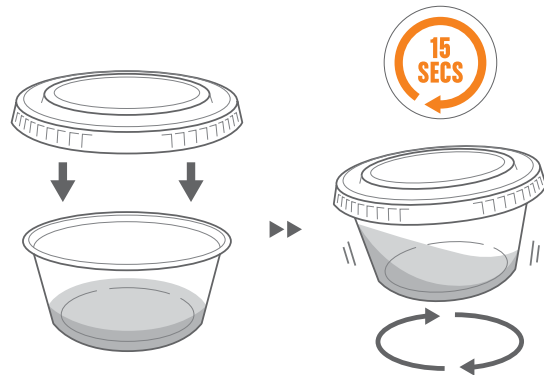


★ Refer to pages 7-8 of Owner's Guide for sample collection guidelines.

**02**

**Prep the Sample**

Wait 30 minutes for sample to liquefy. Just before loading, swirl cup again for **at least 15 seconds**.

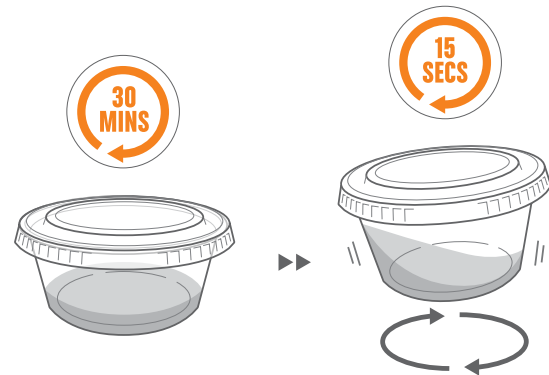


1.888.241.2476 | [support@trakfertility.com](mailto:support@trakfertility.com)

# 03

## Load the Sample

Add **one** full dropper load of sample to Prop. Seal Prop.


TQR-0-1 RO

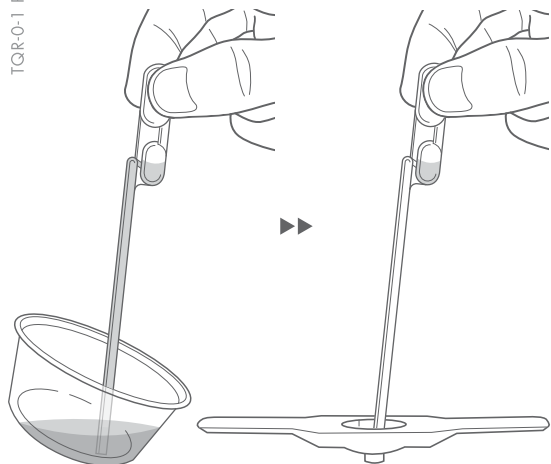

# 04

## Spin the Prop

Attach Prop to metal Engine rod. Close lid. The Engine will stop automatically in about 6 minutes.

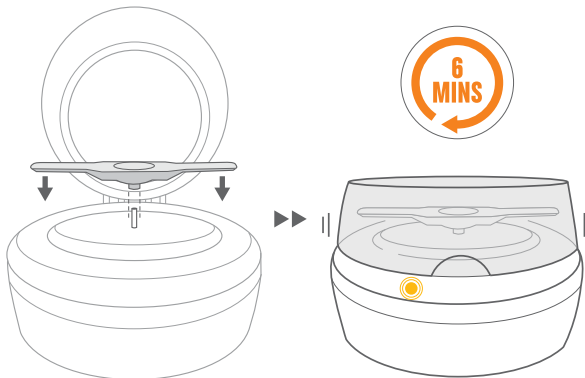


If you see a red light, or there is no light, refer to Owner's Guide for further instructions.

# 05

## Read Your Test Result

Remove Prop and place on a flat surface. Look straight down to read your result.

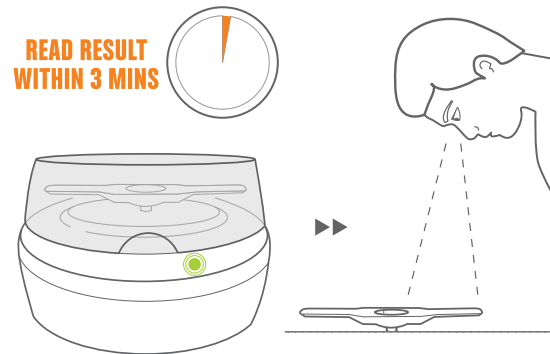

Refer to Owner's Guide to read and understand your result. See pages 19-20 for possible errors. See pages 21-22 for details on reading your result.